




Spare part list – EN

Molift Smart 150

molift®
by Etac

SPL09501 - 2025-12-03 - Rev. 6.0

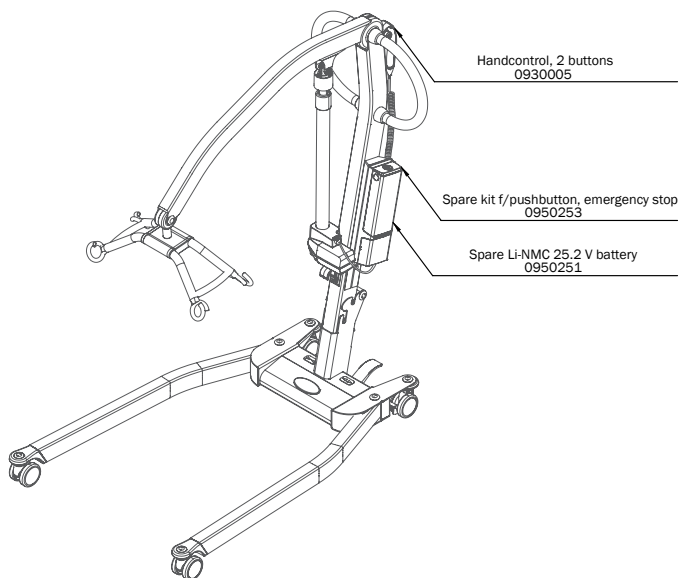

www.etac.com



Fig.1

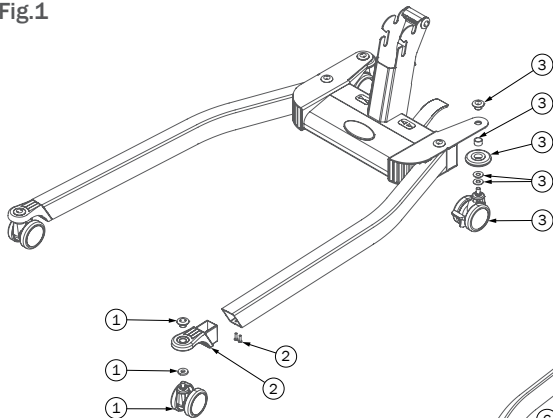


Fig.2

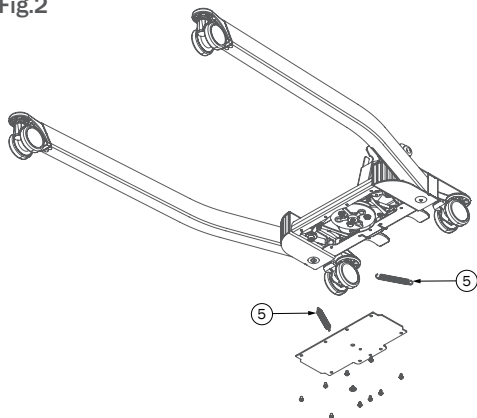


Fig.3

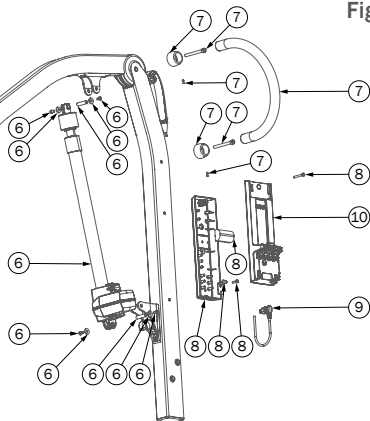


Fig.	Pos.	Title	Item no.
1	1	Front wheel ø75 kit Smart 150/Mover 180	0950201
	2	Front wheel bracket kit Smart 150	0950203
	3	Rear wheel ø75 kit w.brake	0950202
	4	Special tool Molift Smart 150/Mover/Partner 255 (no illustration)	0920501
2	5	Spring kit leg mech. Smart 150	0950250
3	6	Actuator kit Smart 150	0950228
	7	Spare handle kit Smart 150	0950237
	8	Control box mounting kit	0950247
	9	Actuator cable 460 mm	0930116
	10	Spare control box, BLE	0950252
	10	Spare ctrl box w/o chrg. BLE (only available for Japan)	0950252-J
	11	Cable RJ50 f/handctrl. 2&4 btn (no illustration)	0950248
	12	Elastic band ø6x2300mm black (no illustration)	0910174
	13	Clips Smart (no illustration)	0920239

Fig.4

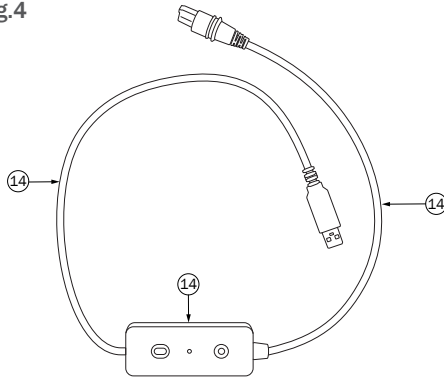


Fig.5

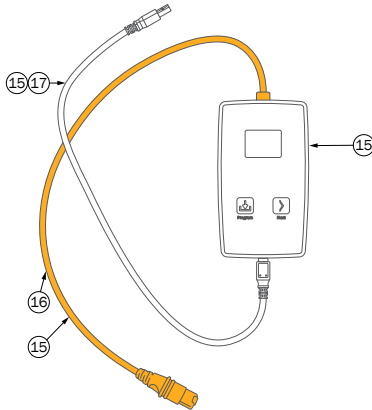


Fig.7

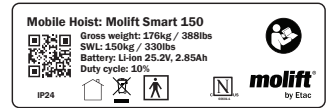


Fig.8

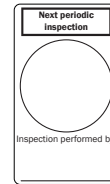


Fig.9



Fig.10



Fig.11



Fig.12

U In : 120-240 V AC, 50-60 Hz
 I In : Max. 350 mA, P In: 35-55 VA

Fig.	Pos.	Title	Item no.
4	14	Cable for Molift Service Tool (USB/RJ50)	2510250
	15	Prog.box kit f/ BLE control box (only for service technician)	0950254
5	16	Spare cable f/ prog. box, RJ50 (only for service technician)	0950152
	17	USB A to USB B cable 1M (only for service technician)	0950153
	18	Spiral cable USA plug (no illustration)	2920092
6	19	Spiral cable EU plug (no illustration)	2920093
	20	Spiral cable GB plug (no illustration)	2920094
	21	Spiral cable AU plug (no illustration)	2920095
7		Product label	1860111
8		Label Safety Check	1100303
9		Label Safety Check	1110306
10		Label Molift by Etac	0950128
11		Label charging instructions (not available for Japan)	0950123
12		Label elec.data f/ BLE ctrl.box (not available for Japan)	0950124



Etac Ltd.

Unit 60, Hartlebury Trading Estate,
Hartlebury, Kidderminster,
Worcestershire, DY10 4JB
+44 121 561 2222



Promefa SA

Avenue de Morges 33
1027 Lonay, CH
+41 44 872 97 79



Etac A/S

Parallevej 3
DK-8751 Gedved



+45 796 858 33



patienthandling@etac.com



www.etac.com

molift[®]
by Etac